






## Auto Image Paste Quick Guide

*Definium\* 8000 / Optima\* XR640 / Discovery\* XR650 / Discovery XR656 and Discovery XR656 Plus*

1. Position and lock the patient positioner
2. Select patient from worklist or manually add patient and start exam
3. Select "Image Paste" category
4. Select "Spine" or "Legs" exam; choose appropriate view
5. Select patient appropriate size
6. **Determine Center of Interest at spine or at posterior lateral malleolus for leg paste; determine Center of Interest for the anatomical area by utilizing the measuring device on the patient positioner**



7. Enter "Center of Interest" on the tube display
8. Define top of the image field with laser: **Angle the tube** so the laser line is at the top of the anatomy of interest
9. Press "Head" on the collimator display 
10. Define bottom of the image field with laser: **Angle the tube** so the laser line is at the bottom of the anatomy of interest
11. Press "Foot" on the collimator display 
12. Press "Set" on the collimator display 

**NOTE: Collimation must be completed prior to selecting "Set"**

13. If collimation not completed prior to selecting "Set" the reset button should be selected; repeat steps 7-12



14. On the acquisition screen verify the number of images that will be acquired

**Image Status: 0/3 COL:5 cm**

15. Make exposures (**press and hold the exposure switch until all exposures are completed**)

**All final paste images should be reviewed by a qualified health care professional for alignment and measurement accuracy before being used for diagnosis.**

*Should the acquired images require repasting, the following steps should be followed:*


### Steps for re-pasting image

1. On the Image Tools Panel click the image processing tab.



2. Click the Repaste Button.

3. Repaste screen displays

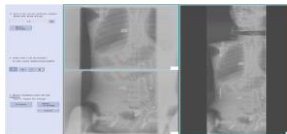

4. Select a region on the drop down list




5. Click and drag in a red cross hair to move it

6. Place cross hair at same anatomical reference point within the image



7. Click **preview** to check registration

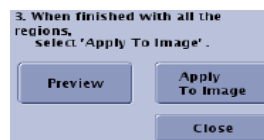

8. When satisfied with image alignment click **apply to image**

9. Click **close** to leave the manual re-paste screen and return to the image viewer

**Auto Image Paste is only available for Definium 8000, Optima XR640, Discovery XR650, Discovery XR656 Discovery XR656 Plus product lines.**