




XR: Manually Transfer Images to a Selected Network/PACS destination


FlashPad™ Digital Upgrade (Wireless Digital Upgrade Option for Proteus XR/a™ and Precision™ 500D)

This reference guide is used to manually transfer images to a Selected Network/PACS destination.

1. Highlight the correct exam

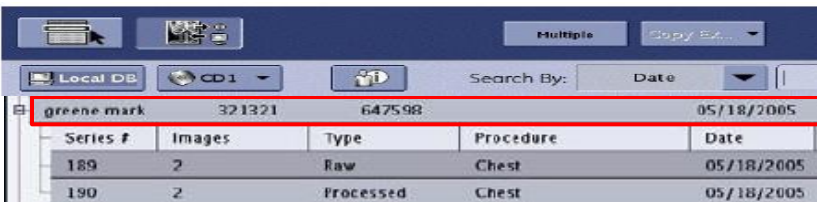

2. Select the correct processed images and manually transfer images to the desired Selected Network/PACS destination

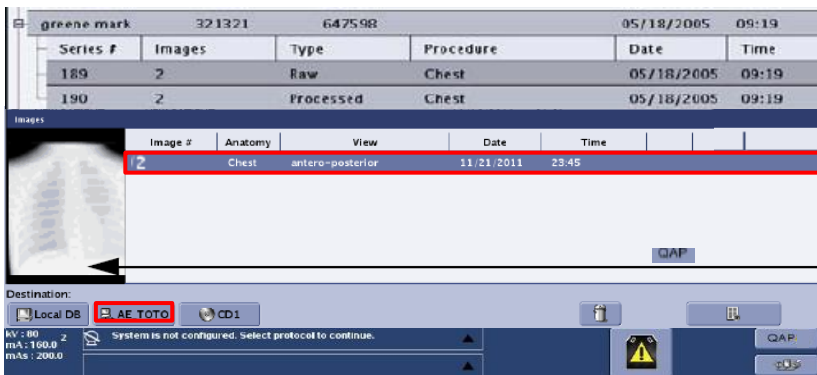

3. Ensure the correct images have transferred to the Selected Network/PACS destination by confirming at the Selected Network/PACS or system transfer log. Clicking on the transfer log icon the log will open display the current transfer status

